

# CORK

MEDICAL

## 24" Extension Tubing

Part Number: CSP-100-ES-EXT

### USE

Cork Medical 24" Extension Tubing is used to add length to port pad tubing and will aid in increasing mobility for patients with a wound located on distal portions of the body.

### INDICATIONS

Cork Medical 24" Extension tubing is indicated for:

- Single wounds where the patient needs longer tubing to increase mobility.

### CONTRAINDICATIONS

Cork Medical 24" Extension Tubing is contraindicated for:

- Do not use the extension tubing with a Y-Connector.
- The increase in tubing length poses a fall hazard.

### WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse or reprocess this tubing. This device is for single use only. Reuse or reprocessing may damage the tubing and may result in injury, infection, or death.
- Consider the extension tubing as an extension of the port pad tubing; tubing should be changed with every dressing change.
- Do not use if the individual package is opened or if the package or the tubing is damaged. Carefully inspect the tubing for damage or defects prior to use.
- Disposal of used product should follow facility policies or local ordinances relating to the handling of potentially infected or bio-hazardous materials.

### DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Inspect the packaging for damage or defects. Replace the entire tubing unit if the packaging is damaged or expired.
3. Verify physician's order for NPWT.
4. Open the dressing package.
5. Clamp port pad tubing.
6. Clamp canister tubing.
7. Connect extension tubing between canister tubing and port pad tubing.
8. Power on the Nisus.
9. Unclamp the canister tubing.
10. Unclamp the port pad tubing.
11. Ensure seal integrity – dressing should have wrinkle appearance.

### STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



This product is non-sterile



Single use only



Do not use if package is damaged

## RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



Manufactured by:

Cork Medical  
8000 Castleway Drive  
Indianapolis, IN 46250 USA