

# CORK

MEDICAL

## Cork Medical Cleanse Kit

Part number: **CLEANSE-KIT-A-18**

### USE

Cork Medical Cleanse Kit is used to transmit negative pressure from the Nisus pump device to the wound base.

### INDICATIONS

The Cork Medical Cleanse Kit is indicated for:

- Use with Nisus NPWT pump device only.
- Use with Nisus NPWT dressing kit in the wound base.

### CONTRAINDICATIONS

The Cork Medical Cleanse Kit is contraindicated for:

- Malignancy of the wound
- Untreated osteomyelitis
- Non-enteric and unexplored fistula
- Necrotic tissue that is greater than 20 percent of wound bed

### WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse, reprocess or re-sterilize this product. This device is for single use only. Reuse, reprocessing or re-sterilization may damage the product and may result in injury, infection, or death.
- Do not use if the individual package is opened or if the package or port pad is damaged. Carefully inspect the product for damage or defects prior to use.
- Disposal of used port pad should follow facility policies or local ordinances relating to the handling of potentially infected or bio-hazardous materials.

### DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Inspect the packaging for damage, defects and ensure the product is not expired. Replace the entire unit if the packaging is damaged.
3. Verify physician's order for NPWT and ensure completeness of order prior to starting and review the contraindications list above.
4. Open the port pad.
5. NPWT Dressing is in place.
6. Gently lift the drape with your thumb and forefinger and cut a 2 cm round hole on the top of the drape. (Do not slit drape because the slit will reseal itself when NPWT is on.)
7. Make sure to properly align the port pad opening with the cut hole on the drape. After properly aligning the holes, peel off the bottom liner and attach port pad to the drape. Once finished, peel off the top liner and the side blue tabs and remove. Pat around the port pad to ensure the dome is securely in place.
8. Connect the tubing to the canister tubing via the luer lock method.
9. Prior to instilling any liquid substance in the cleanse port, cleanse the cleanse port with alcohol wipe.
10. Power off Nisus device.
11. Clamp the port pad tubing 2 inches from dome of port pad.

12. Do not leave Nisus device powered off for more than 2 hours.
13. Attach luer lock syringe to cleanse port and manually inject solution, per physician's order at a rate not exceeding 5ml per second.
14. Never infuse more than 60 ml into wound base via cleanse port.
15. Once solution is infused remove provided syringe and place in a secure location.
16. Unclamp port pad tubing.
17. After prescribed dwell time of solution, power back on Nisus device and resume NPWT at prescribed therapy, per Physician's order.

### STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



Single use only, do not resterilize



Do not use if package is damaged

STERILE EO STERILIZED WITH ETHYLENE OXIDE

### RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



Manufactured by:

Cork Medical  
8000 Castleway Drive  
Indianapolis, IN 46250 USA