

# CORK

MEDICAL

## Medium Foam

Part Number: CWF-100-ES-B

## Large Foam

Part Number: CWF-200-ES-B

## USE

Cork Medical's foam should be used in conjunction with the Nisus NPWT System only. The foam is used to dress the wounds per the physician order.

## INDICATIONS

The foam is indicated for:

- Full thickness injuries
- Skin grafts
- Partial thickness burns
- Skin flaps
- Incisional line management

## CONTRAINDICATIONS

The foam is contraindicated for:

- Malignancy of the wound
- Untreated osteomyelitis
- Non-enteric and unexplored fistula
- Necrotic tissue that is greater than 20 percent of wound bed

## WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse, reprocess or re-sterilize this product. This device is for single use only. Reuse, reprocessing or re-sterilization may damage the product and may result in injury, infection, or death.
- Do not use if the individual package is opened or if the package or the foam is damaged. Carefully inspect the product for damage or defects prior to use.
- Disposal of used product should follow facility policies or local ordinances relating to the handling of potentially infected or bio-hazardous materials.

## DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Inspect the packaging for damage or defects; ensure product is not expired. Replace the entire unit if the packaging is damaged or expired.
3. Verify physician's order for NPWT and ensure completeness of order prior to starting and review the contraindications list above. Cleanse the wound and periwound area according to the institution guidelines and physician's order.
4. Open the dressing package.
5. Cut the black foam to the shape of the wound and ensure that the entire wound will be covered with foam. (Always cut the foam away from the wound to prevent any unwanted debris from falling into the wound bed.) Ensure that no foam comes in contact with the periwound area.

## STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



Do not use if package is damaged



Single use only, do not re-sterilize

STERILE EO STERILIZED WITH ETHYLENE OXIDE

## RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



Manufactured by:

Cork Medical

8000 Castleway Drive

Indianapolis, IN 46250 USA