

# CORK

MEDICAL

## Opitfoam White Foam Wound Dressing

Catalog Number: FOAM-WHITE-A-16

Opitfoam white foam wound dressing is a sterile, absorbent, polyurethane foam wound dressing.

### USE

Opitfoam white foam wound dressing absorbs wound drainage, promotes a moist environment helpful to healing, and serves as a barrier against water and bacteria when used as a secondary dressing.

### INDICATIONS

- Opitfoam white foam wound dressing is indicated for minor abrasions, minor lacerations, and minor scalds and burns.
- When under the care or direction of a licensed practitioner they are indicated for:
  - Dermal ulcers (e.g., pressure ulcers, leg ulcers, and other secreting lesions)
  - Second degree burns
  - Donor sites
  - More severe lacerations, cuts and abrasions
  - Other superficial wounds

### CONTRAINDICATIONS

- Opitfoam white foam wound dressing is contraindicated for:
  - Heavily bleeding wounds
  - Individuals who are sensitive to or have had an allergic reaction to the dressing or its materials
  - Use during surgical procedures as a surgical sponge

### WARNINGS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse, reprocess, or re-sterilize this dressing. This device is for single use only. Reuse, reprocessing, or re-sterilization may damage the dressing, and may result in injury, infection, or death.
- Do not use if the individual package is opened or if the package or the dressing is damaged. Carefully inspect the dressing for damage or defects prior to use.
- If you have any signs of an allergic reaction, discontinue using the dressing and rinse the wound and surrounding skin.
- If applicable, ensure the dressing adheres to skin.
- If dressing is ineffective or does not absorb, discard and replace.
- If applicable, dispose of all used dressings with care to avoid exposure to or contamination of biohazard.

### CAUTIONS

- Ensure the dressing fully covers your wound.
- If the dressing rips or tears prior to or during application, discard the torn dressing and replace it.
  - Prior to removal, ensure the dressing has not adhered to the wound. If the dressing has adhered to the wound, rinse with water or saline solution to loosen.
  - If applicable, take proper care when removing a dressing with adhesive as it may result in skin irritation. If the adhesive is too strong or the skin is fragile, rinse with water or saline solution to loosen.

### DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Inspect the packaging for damage or defects. Replace the entire dressing if the packaging is damaged.
3. Clean the wound of excess drainage and fluid, and thoroughly dry skin around the wound.
4. Open the dressing package.

### NON-ADHESIVE, BORDERLESS DRESSINGS

1. Place the foam in the wound bed.
2. Cover with an appropriately sized secondary dressing to keep the wound moist and the foam in place.
3. Change the dressing as needed when wet, soiled, leaking, or as directed by a licensed practitioner.
4. To remove, first remove the secondary dressing. Next, remove the foam dressing from the wound.
5. Dispose of the soiled dressing.

### STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.

### WARRANTY

Products are warranted for ninety (90) days from the date of shipment from Cork Medical as to product quality and workmanship. **CORK MEDICAL WRITTEN WARRANTIES ARE GIVEN IN LIEU OF ANY IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**


STERILE EO STERILIZED WITH ETHYLENE OXIDE



Do not use if package is damaged

### RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).

 **Manufactured for:**  
Cork Medical  
8050 Castleway Drive  
Indianapolis, IN 46250 USA