

CORK

MEDICAL

Port Pad

Part Number: CSP-100-ES-C

USE

Cork Medical Port Pad assembly is used to transmit negative pressure from the Nisus pump device to the wound base.

INDICATIONS

The Cork Port Pad Assembly is indicated for:

- Use with the Nisus NPWT pump device only.
- Use with the Nisus NPWT dressing kit in the wound base.

CONTRAINDICATIONS

Cork Port Pad Assembly is contraindicated for:

- Malignancy of the wound
- Untreated osteomyelitis
- Non-enteric and unexplored fistula
- Necrotic tissue that is greater than 20 percent of wound bed

WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse, reprocess or re-sterilize this product. This device is for single use only. Reuse, reprocessing or re-sterilization may damage the product and may result in injury, infection, or death.
- Do not use if the individual package is opened or if the package or the port pad is damaged. Carefully inspect the product for damage or defects prior to use.
- Disposal of used port pad should follow facility policies or local ordinances relating to the handling of potentially infected or bio-hazardous materials.

DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Inspect the packaging for damage, defects and ensure the product is not expired. Replace the entire unit if the packaging is damaged.
3. Verify physician's order for NPWT and ensure completeness of order prior to starting and review the contraindications list above.
4. Open the port pad.
5. NPWT Dressing is in place.
6. Gently lift the drape with your thumb and forefinger and cut a 2 cm round hole on the top of the drape. (Do not slit drape because the slit will reseal itself when NPWT is on.)
7. Make sure to properly align the port pad opening with the cut hole on the drape. After properly aligning the holes, peel off the bottom liner and attach port pad to the drape. Once finished, peel off the top liner and the side blue tabs and remove. Pat around the port pad to ensure the device is securely in place.
8. Connect the tubing to the canister tubing via the luer lock method and start the NPWT per physician's order.

STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



Do not use if package is damaged



Single use only, do not re-sterilize

STERILE  STERILIZED WITH ETHYLENE OXIDE

RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



Manufactured by:

Cork Medical

8000 Castleway Drive

Indianapolis, IN 46250 USA