

CORK

MEDICAL

Cork 1-2-3 Drape Film Dressing 10"x12"
Part Number: CWD-200

USE

Cork Medical's drape should be used in conjunction with the Nisus NPWT System only. The drape is used to seal the wounds per the physician order.

INDICATIONS

The drape is indicated for:

- Create an occlusive environment when used with Cork Medical's NPWT system
- Used in conjunction with Cork Medical's NPWT foam and port pad

CONTRAINDICATIONS

The drape is contraindicated for:

- Malignancy of the wound
- Untreated osteomyelitis
- Non-enteric and unexplored fistula
- Necrotic tissue that is greater than 20 percent of wound bed
- Patient has a known sensitivity to drape
- Drape is not intended to be placed directly into wound base

WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse, reprocess or re-sterilize this product. This device is for single use only. Reuse, reprocessing or re-sterilization may damage the product and may result in injury, infection, or death.
- Do not use if the individual package is opened or if the package or the product is damaged. Carefully inspect the product for damage or defects prior to use.
- Disposal of used drape should follow facility policies or local ordinances relating to the handling of potentially infected or bio-hazardous materials.

DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Inspect the packaging for damage or defects; ensure product is not expired. Replace the entire unit if the packaging is damaged or expired.
3. Verify physician's order for NPWT and ensure completeness of order prior to starting and review the contraindications list above. Cleanse the wound and periwound area according to the institution guidelines and physician's order.
4. Open the dressing package.
5. Peel off tab #1 first. Gently press down the exposed adhesive on the drape to the skin. (Cutting the drape is recommended for better drape control and to avoid any mistakes.) Peel off #2 and gently press drape around the periwound. Apply gentle pressure to the center of the drape and work outward from the center area. (Do not compress foam dressing while applying the drape.) Make sure to form a seal before peeling off #3.

STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



Do not use if package is damaged



Single use only, do not resterilize

STERILE R

RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



Manufactured by:

Cork Medical
8000 Castleway Drive
Indianapolis, IN 46250 USA