



## Y-Connector

**Part Number: CWY-100**

### USE

The Y-Connector is designed to simultaneously treat multiple wounds on the same patient. The multiple wounds should be the same etiology requiring the same pressure settings.

### INDICATIONS

The Nisus Y-Connector is indicated for:

- Compatible with all Nisus NPWT wound dressings.

### CONTRAINDICATIONS

The Nisus Y-Connector is contraindicated for:

- It is not recommended to Y-Connect grafts or flaps.
- It is not recommended to use more than one Y-Connector per therapy unit.
- Do not connect infected wounds with non-infected wounds through a Y-Connector.
- Do not connect wounds with different etiologies in which cross contamination may occur.
- Avoid using Y-Connector to connect wounds that would be optimally treated with different pressure settings.

### WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse or reprocess. This product is for single use only. Reuse or reprocessing may damage the product, and may result in injury, infection, or death.
- Consider the Y-Connector an extension of the canister tubing and change Y-Connector with every canister change.
- Do not use if the individual package is opened or if the package or the product is damaged. Carefully inspect the product for damage or defects prior to use.
- Disposal of used Y-Connector should follow facility policies or local ordinances relating to the handling of potentially infected or bio-hazardous materials.

### DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Inspect the packaging for damage or defects. Replace the entire product if the packaging is damaged.
3. Verify the physician's order for NPWT.
4. Open the Y-Connector package.
5. Dress both wounds with NPWT dressing kit, per physician order.
6. Connect the Y-Connector to canister tubing at the luer lock at the bottom of the Y-Connector.
7. Connect each port pad tubing to the Y-Connector, clamp both port pad tubing.
8. Power on Nisus pump, unclamp the distal wound's port pad and clamp.
9. Unclamp proximal wound's port pad clamp
10. Ensure seal integrity – dressing should have a wrinkle appearance.

### STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



This product is non-sterile



Single use only



Do not use if package is damaged

### RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



**Manufactured by:**  
Cork Medical Products  
8000 Castleway Drive  
Indianapolis, IN 46250 USA