



**250 ml Canister**  
**Part Number: CPC-250**

**500 ml Canister**  
**Part Number: CPC-500**

#### USE

The canister collection system is used to collect exudate and debris from the wound base.

#### INDICATIONS

Use canisters in conjunction with the Nisus NPWT System. The 250 ml canister should be utilized for small exuding wounds, and the 500 ml canister should be utilized for large exuding wounds.

#### CONTRAINDICATIONS

Do not use the Nisus canisters on any other NPWT device.

#### WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse or reprocess this product. This device is for single use only. Reuse or reprocessing may damage the product which may result in injury, infection, or death.
- Do not use if the individual package is opened or if the package is damaged. Carefully inspect the canister for damage or defects prior to use.
- Disposal of used canisters should follow facility policies or local ordinances relating to the handling of potentially infected or bio-hazardous materials.

#### DIRECTIONS FOR USE

1. Carefully read the **WARNINGS** and **CAUTIONS** prior to use.
2. Inspect the packaging for damage and defects. Replace the unit if the packaging is damaged or expired.
3. Verify physician's order for NPWT.
4. Open the canister dressing.
5. Clamp canister tubing.
6. Hinge canister in place on the back of the Nisus device.
7. Ensure that you hear a 'click'.
8. Gently tug on the canister to ensure it is latched in place.
9. Connect canister tubing to port pad tubing.
10. Ensure canister clamp and port pad tubing is unclamped.
11. Power on the Nisus device.
12. Ensure seal integrity – dressing should have wrinkle appearance.

#### STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



This product is non-sterile



Single use only



Do not use if package is damaged

#### RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



Manufactured for:  
Cork Medical Products  
8000 Castleway Drive  
Indianapolis, IN 46250 USA