



Nisus Wire Hanger

Part Number: CMP-9GA

USE

The Nisus NPWT Pump is designed to rest safely inside the Nisus Wire Hanger. Recommended for bedside and wheelchair use.

INDICATIONS

The Nisus Wire Hanger is indicated for:

- Use with the Nisus NPWT Pump.

CONTRAINDICATIONS

The Nisus Wire Hanger is contraindicated for:

- Not for use with any other device, medical or personal.

WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not use if the product is damaged. Carefully inspect the product for damage or defects prior to use.
- Only use the wire hanger provided by Cork Medical Products.
- Always adhere to your facility policies when cleaning medical equipment.

DIRECTIONS FOR USE

1. Place hooks of the hanger onto rigid ledge.
2. Orient the back of basket against the rigid surface (e.g. foot of bed).
3. Lower Nisus unit into hanger so screen is facing outward.

STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



Manufactured For:
Cork Medical Products
8000 Castleway Drive
Indianapolis, IN 46250 USA