



Nisus Battery Charger

Part Number: NIS-CHRG-C-16

USE

The Nisus NPWT Pump is designed to run on a lithium-ion rechargeable battery. The pump can also operate with the provided battery charger plugged in.

INDICATIONS

The Nisus Battery Charger is indicated for:

- Use with the Nisus NPWT Pump.

CONTRAINDICATIONS

The Nisus Battery Charger is contraindicated for:

- Not for use with any other device, medical or personal.

WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not use if the individual package is opened or if the package or the product is damaged. Carefully inspect the product for damage or defects prior to use.
- Only use the battery charger provided by Cork Medical Products.
- Always adhere to your facility policies when cleaning medical equipment.

DIRECTIONS FOR USE

1. Ensure rechargeable power cord is securely inserted into the power port on the Nisus device.
2. Connect the opposite end of the AC electrical cord into the wall or other appropriate electric outlet.
3. Remove charging cord once Nisus device is fully charged.

STORAGE AND TRANSPORT CONDITIONS

Store at room temperature. Avoid excessive heat and humidity.



Manufactured For:

**Cork Medical Products
8000 Castleway Drive
Indianapolis, IN 46250 USA**